
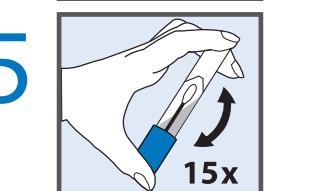
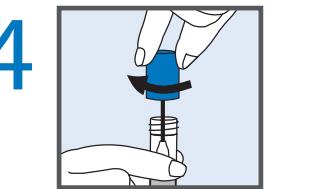
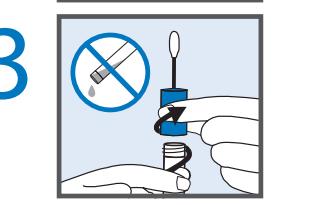
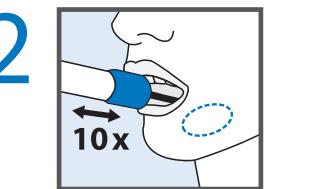
ORACollect™-DNA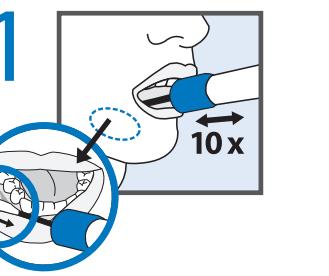

DNAgenotek™

REF OCR-100
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America):
1.866.813.6354

+1.613.273.5757

info@dnagenotek.com

www.dnagenotek.com

Patent (EN) Brevet (FR) Patent (ES)
Patent (DE) Patent (NB) Patent (SV)
Patent (DA) Patent (NL) Patent (CS)
(www.dnagenotek.com/legalnotices)

ENGLISH User instructions

Read all instructions prior to collection.

Collection precautions:

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting oral sample.

Intended use: This product is designed for the collection and stabilization of DNA from human oral samples

Contents: 1 kit containing 1 mL stabilizing liquid

Warnings and precautions: Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C - 25°C (59°F - 77°F)

Summary and explanation of the kit: ORACollect™-DNA is a self-collection kit that provides the materials and instructions for collecting human oral samples.

Instructions for sample collection:

- Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gum (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.
- Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.
- Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.
- Turn the cap upside down, insert the sponge into the tube and close cap tightly.
- Invert the capped tube and shake vigorously 15 times.

Label legend:

REF	Catalog number
CE	CE marking
UKCA	UKCA marking
MD	Medical device
②	Do not reuse
CONSULT	Consult instructions for use
Caution	
Use by	
LOT	Lot number
EC REP	European Authorized Representative
CH REP	Swiss Authorized Representative
UDI	Unique Device Identifier
QTY	Quantity of units per package
15°C - 25°C 59°F - 77°F	Storage instructions
Made in US FROM FOREIGN AND DOMESTIC MATERIALS	Made in US from foreign and domestic materials

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OraSur Technologies, Inc., all rights reserved.